

SPEARHEAD®

Guide to Spearhead Slings

Spearhead offer a full range of Slings specially selected to meet the diverse and varied requirements of users in care homes.

STEP 1: TYPE OF HOIST & SLING

Are you using a hoist or standaid to lift the user?

For Hoists: Do you require a sling for transfer purposes and/or toileting?

For Standaid: Choose between normal standing sling, or if you're using a standaid sling for user transfer, you may want to consider the transport sling

STEP 2: TYPE OF FIXING

Identify the type of fixings your hoist or standaid has. These fall into two types: loop and clip fixing. When selecting your sling ensure it has the right fixing for your hoist or standaid. If you're unsure please speak with one of our trusted advisors.




LOOP FIXING



CLIP FIXING

STEP 3: TYPE OF MATERIAL

All Spearhead slings held in stock are supplied in one of these fabrics:

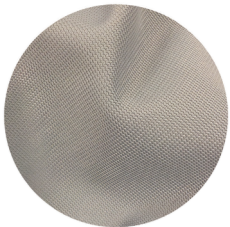

POLYESTER

Standard fabric used for all toileting slings



MESH POLYESTER

Standard grade fabric that is ideal for bathing and showering

STEP 4: CHOOSE YOUR SIZE

Select the size required using the following sizing chart.

Sling Size	Girth*	Height
Extra Small	61-76cm	114-132cm
Small	68-91cm	129-150cm
Medium	79-104cm	147-168cm
Large	102-129cm	165-178cm
Extra Large	124-150cm	170-183cm

*Women: around the hips. Men: around the chest.

- ▲ Unique embroidered label identifying safe working load and serial no
- ▲ High temperature compatible with launderable labels (85°C/185°F)
- ▲ LOLER service labels to quickly identify the current service status of each sling
- ▲ 220kg weight capacity
- ▲ Flame retardant fabric
- ▲ 1 year warranty

STEP 5: ORDER

Once you have followed the above 4 steps you will be ready to order the correct sling. If you are unsure on any of the steps please speak with one of our trusted advisors.