



INSTRUCTIONS FOR DISINFECTION USING BLEACH SOLUTIONS OF SPD1374 BLEACH TABLETS*

HOW TO USE:

STEP 1

Wear appropriate PPE: Gloves, Apron and Safety Glasses. If in a confined space, open a window for ventilation.

STEP 2

Fill the bottle with water to shoulder height for 1 litre volume.

STEP 3

Drop in the required number of tablets and allow to dissolve, shaking the bottle prior to use.

STEP 4

Apply the solution liberally over the surface using the push-pull cap or trigger spray head.

STEP 5

Keep prepared solution for no longer than 24 hours. Dispose of residual solution into an appropriate drain, sluice or sink with running water.



GENERAL DISINFECTION: SANITISING SOLUTION FOR PRE-CLEANED SURFACES

Dissolve 1 tablet in 1 litre warm water for 1,000 ppm available chlorine. Apply solution using the push-pull cap or spray and allow 5 minutes contact time on fully wetted surface. For disinfection of sporicidal C. Difficile allow 10 minutes contact time.

DISINFECTION OF BLOOD SPILLAGES

Dissolve 10 tablets in 1 litre warm water for 10,000 ppm available chlorine. Cover spillage with paper towels. Apply solution using the push-pull cap. Allow 3 minutes contact time on fully wetted surface. Remove and dispose of spill material to clinical waste. Clean the area with a neutral detergent.

SPD1374 tablets have undergone extensive testing against:

- EN13697 – Fungicidal Activity
- EN13704 – Sporicidal Activity
- EN14476 – Coronavirus
- EN1276 – Bactericidal Activity
- EN1650 – Fungi & Yeast

*3.25g tablets containing 1.67g NaDCC. Tablets do not contain detergent.

SPEARHEAD®

4 Symington Place
Riverside Business Park
Irvine
KA11 5DE

0345 180 1800
sales@spearheadhealthcare.com
www.spearheadhealthcare.com

TAKING CARE OF CARE